



**A custom-made
solution !**

- **A comfortable hearing protection for a better communication in a noisy environment !**

Advantages

- A very compact custom-made hearing protection that fits perfectly in the user's auditory canal
- 100% made in Switzerland
- Comfortable to wear as it is made entirely of soft, flexible silicon material that causes no pressure points in the ear canal
- Protects against loud or extreme noises

Advantages

- Small and ergonomic
- Uses an adapted noise filter that prevents from an excessive attenuation. The device is provided with a RC15 filter.
- Ensures a better speech understanding and warning signals can still be heard
- Offers a flat attenuation in high frequencies
- A cord is attached to the tiny grip to make it very easy to insert and remove

Attenuation Characteristics


	Frequency (Hz)						
Filter	125	250	500	1000	2000	4000	8000
RC 19	22.4	23.9	23.7	22.5	22.1	20.4	26.2
RC 18	17.5	19.1	21.4	21.8	21.9	23.3	19.1
RC 17	12.7	15.6	20.3	19.9	23.5	21.2	18.9
RC 15	10.8	12.8	17	17.9	22.9	23.5	17.5
RC 13	7.3	9.5	13.8	16	22.2	22.9	17.1

These broadspectrum filters have very linear attenuation properties.

Filter can easily be changed according to the specific environment requests.

Please contact us directly for individual queries.

Average attenuation

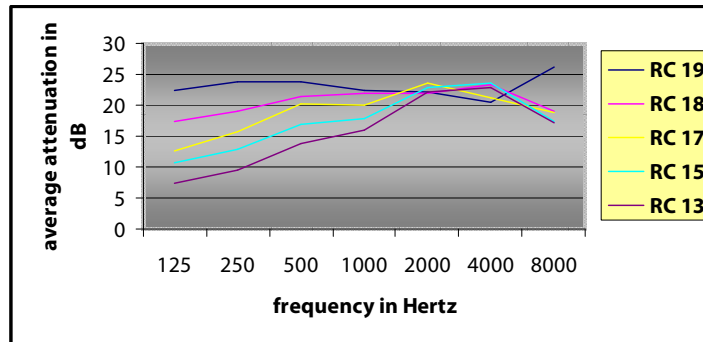

- **Noise** is not only a source of irritation, but primarily a major cause of hearing impairment. Once hearing is impaired it cannot recover and is therefore irreversible. 'Hard of hearing due to noise is one of the three most common occupational diseases. Hearing protection is therefore certainly not an extravagance.

OMARA ClearSound
hearing protection helps to
prevent irreversible hearing loss.

OMARA ClearSound Set

comes with a complimentary set of
accessories

- Cord
- Cerumen stick
- Otoease liquid
- Transportation cover
- Case

User Manual

Please read these instructions carefully before use

- **Inserting the device**
- The protection device is custom-made for the user. The blue inscription is intended for the left ear and the red inscription is for the right ear.
- New devices may not slide easily into your ear the first few times. We therefore recommend applying some Otoease liquid onto the part of the device that is inserted into the auditory channel.

- **Removing the device**
- Grab the device by the grip
- Remove the protection device by pushing it slightly upwards and pulling it from your ear while rotating it towards the front
- **NEVER** remove the device **BY PULLING THE CABLE**. This may cause your eardrum to rupture.

Maintenance

- Clean your OMARA ClearSound hearing protection devices regularly :
 - Remove any earwax with the cerumen stick provided
 - Use a clean cloth with a drop of Otoease to clean the outer part of the device.
 - Avoid applying too much liquid to the filters
 - Dry the protection devices with a cloth before storing them in the protection cover provided
-
- **This product has been manufactured with the utmost care and tailored fully to your ears. However, it can be necessary to make a few small adjustments: if this is the case, please contact us.**

**For more information
please contact**

OMARA

Amplifon SA

Av. de la Gare 43 bis - CP 39

1001 Lausanne / Switzerland

www.omara.ch

info.omara@amplifon.com